Professional Procedure Guide

A1CNow+ Professional
Multi-Test HbA1c System

Note: All instances of the box or pouches in this insert are representative ONLY. Please refer to the packaging included with your specific kit.


Introduction

- If the temperature label, placed on the outside of every kit, is exposed to a temperature in excess of 50°C (122°F), the dot on the label will turn red and the product should not be used.
- Run the test with all parts of the test kit at the same temperature within the specified range.
- If the kit has recently been at high temperatures (above 28°C (82°F)) or in the refrigerator, keep the kit at room temperature for at least one hour before use.
- Avoid running the test in direct sunlight, on hot or cold surfaces, or near sources of heat or cold.
- Quality control materials should be used to confirm the test kit is working properly. Refer to the product insert for information on when to run controls.

28°C (82°F)
18°C (64°F)
Complete test within 15 minutes.


Match Lot Numbers

Use analyzer only with the materials included in the original kit. The analyzer will expire after the programmed number of tests have been run. If another test cartridge is inserted, the analyzer will display "00 TL".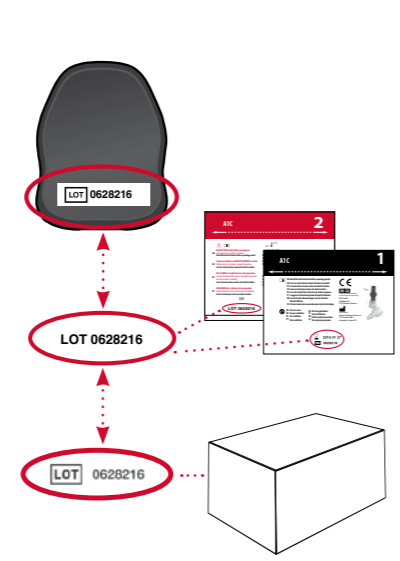

1 Open Plastic Shaker Pouch*

Tear plastic pouch open at the perforation line.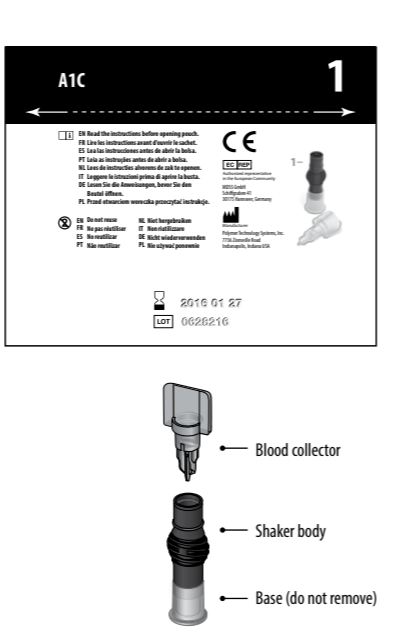

2 Collect Blood


Fingertip method

- Use your own lancet device to draw blood.
- Gently touch blood drop to fill.


OR

Venous draw method

- Mix blood well before testing.
- Collect blood from a slide.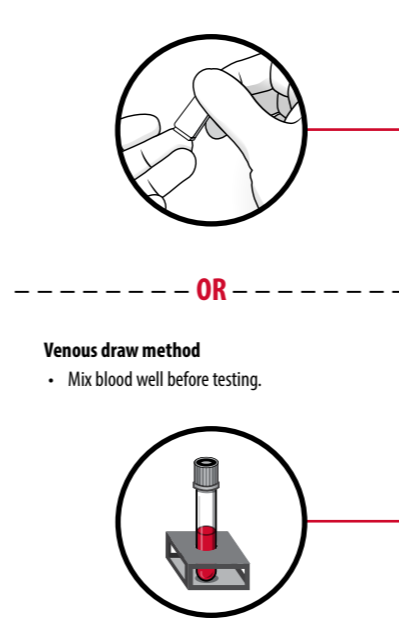


3 Verify Blood Amount

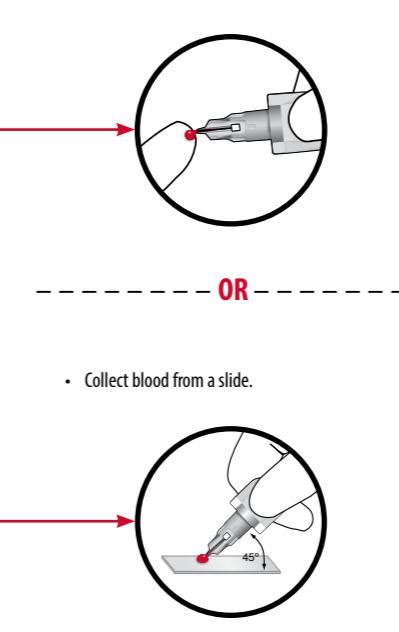Too little: Add more blood
Just right
Too much: Wipe away excess

4 Insert Blood Collector


Fully insert blood collector into shaker body. Twisting motion helps.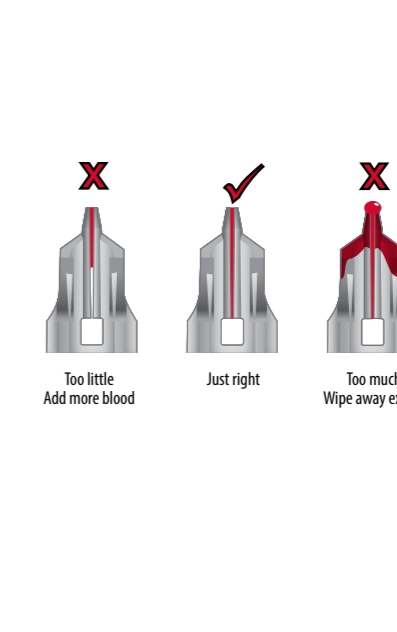

Not fully inserted
Fully inserted

5 Mix


- Shake vigorously 6-8 times. This will mix the blood with the solution.
- Stand shaker on table while preparing cartridge.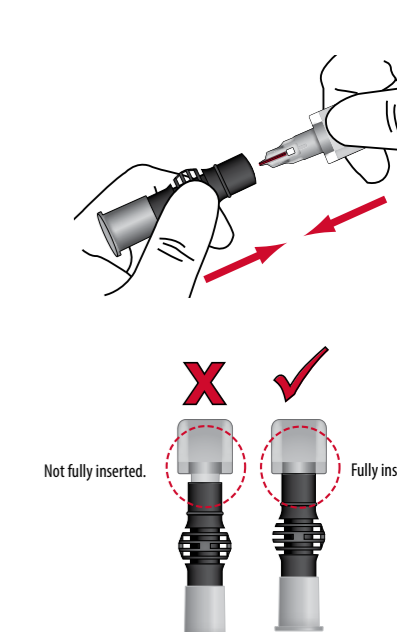

6 Open Foil Cartridge Pouch*


Tear foil pouch open at the notches on the sides.

Caution: Do NOT open foil pouch until this step. For within 2 minutes of opening. If foil pouch is damaged, do not use.

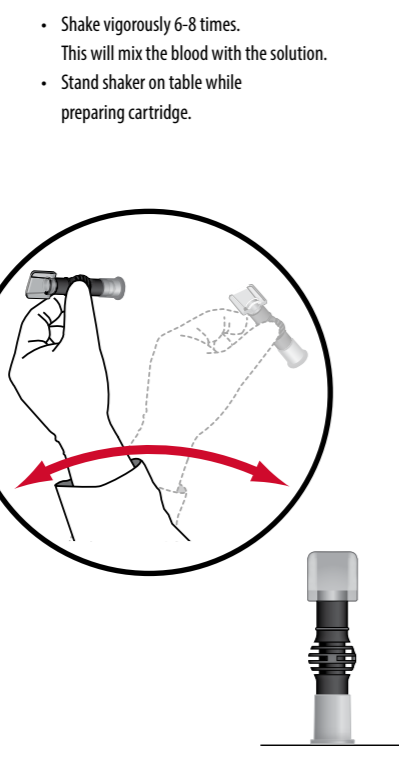7 Insert Cartridge

- "Click" test cartridge into place.
- Analyzer and test cartridge codes must match.
- If codes do not match, call Customer Service at +1-317-870-5610 or your local authorized distributor.

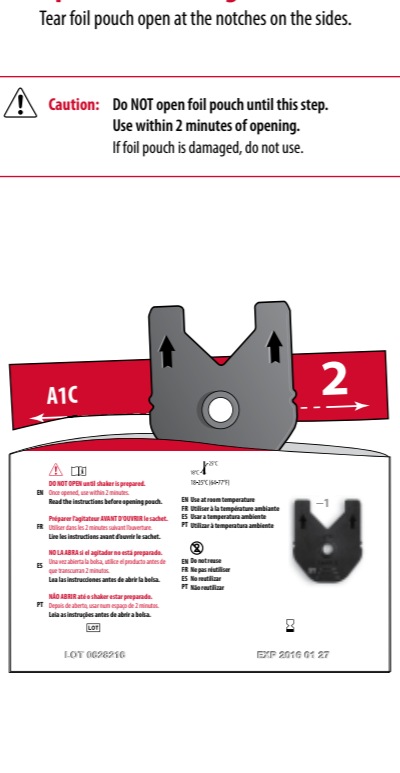
8 Prepare Shaker

Remove shaker base.

9 Dispense Sample into Cartridge

- Ensure analyzer is on level surface.
- Push down completely to dispense diluted sample. Remove quickly.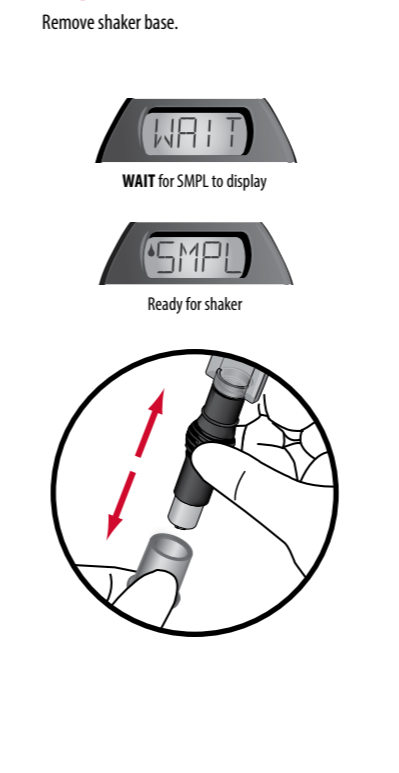

Do not handle analyzer again until test is complete!

10 5 Minutes to Results


Display counts down
OK test result
Result
of tests left

This result cycle remains displayed for 15 minutes or until the next test cartridge is inserted. If "QCOK" is not displayed, please contact Customer Service or consult the troubleshooting section.

11 Dispose of Cartridge / Save Analyzer

- Dispose of in accordance with your biohazard procedures.
- Save analyzer for additional tests. Analyzer will display, for example, "01TL" if there is one test left and "00TL" if you just used your last test.
- Analyzer will show result for 15 minutes and will turn off automatically.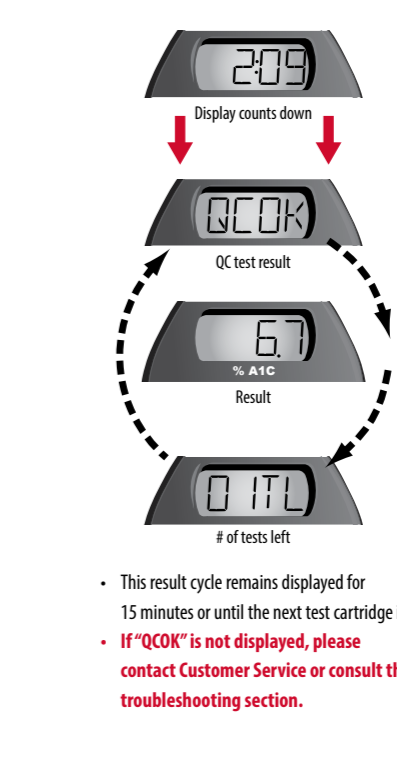

NOTE: To run another test, use a new blood collector, shaker, and test cartridge from the same kit and return to Step 1.

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